

NuLids™

BEFORE YOU START

Confirm your NuLids System registration. This will ensure you are correctly registered as the owner, receive timely product notices and updates, and are able to easily order future supplies of Soft Tips.

AVAILABLE RESOURCES

- Read the User Manual at www.NuSightMedical.com
- Watch videos on “How to Use NuLids” at www.NuSightMedical.com
- Email or call our Customer Solutions team

CHARGING THE HAND PIECE

First Time Use: Charge hand piece fully until solid green light seen.

Future Uses: Re-charge hand piece when amber light flashes. Charging is complete when flashing green light turns solid.

NUSIGHT MEDICAL CUSTOMER SOLUTIONS

Need Help:

Email: CS@NuSightMedical.com (or)

Phone: 1-833-368-5437, ext 1

- Get answers to product usage questions
- Order more NuLids Soft Tips
- Enroll in the automatic refill program
- Share your success story!

WARNINGS & CAUTIONS

Use NuLids only under care of an eye doctor.

Discontinue treatment & consult your eye care professional if discomfort on eye lids or eyeball is noted.

Always use the lubricant gel or cleaner recommended by your eye care professional.

DO NOT re-use Soft Tips. Dispose after a single treatment to avoid infection or injury.

DO NOT share a NuLids Soft Tip with another person.

Keep the hand piece dry and free from excess liquids.

Refer to the User's Guide located at www.NuSightMedical.com for complete instructions for use.

DO NOT put the Soft Tip on your eye or anything other than the eyelids.

DO NOT conduct treatment for more than one 15 second cycle per eyelid, per day.



NuSight Medical[®]

Dry eye relief...In the blink of an eye

18029 Calle Ambiente, Suite 509
Rancho Santa Fe, CA 92067
www.NuSightMedical.com
1-833-368-5437