

# NuLids™

## QUICK START GUIDE\*




## PREPARING FOR USE

- Press On/Off button on hand piece once. Result: White light on tip.
- Align opening on top of hand piece with peg on Soft Tip until it magnetically snaps into position. Confirm the Tip is firmly in place.
- Hold hand piece in dominant hand.
- Apply lubricant gel or cleanser to Soft Tip.



## Treat Both Lower Lids

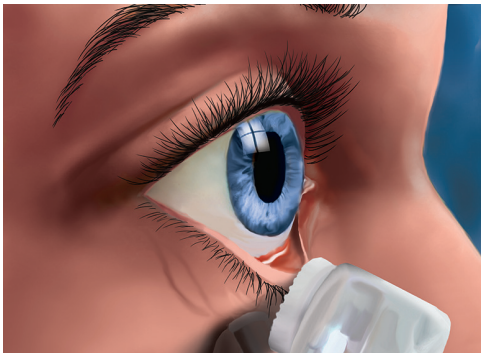

- Select a lower lid. Pull lower lid down gently with opposite hand while looking up.
- Apply Soft Tip to lower lid edge and lash line.
- Move oscillating Soft Tip slowly from one side of lid to other for 15 seconds.
- Hand piece will pause briefly after a 15 second cycle.
- Repeat on opposite lower lid for an additional 15 seconds.
- Hand piece cycles at 15 second intervals until On/Off button is pressed.

## Treat Both Upper Lids



- Select an upper lid. Gently pull lid up while squeezing eyelids closed for access to whole upper lid margin.
- Apply edge of Soft Tip to upper lid edge and lash line.
- Move oscillating Soft Tip slowly from one side to other for one 15 second cycle.
- Repeat on opposite upper lid for final 15 seconds of treatment.
- Press On/Off button to stop device.
- Dispose of used Soft Tip.

## Suggestions

Use a mirror to guide Soft Tip placement. Begin each treatment with clean hands and makeup removed.

Be conscious of where you place the face of the Soft Tip prior to treatment. Unplug hand piece from the USB cord prior to treatment.

Continue with treatment in a non rushed manner when hand piece pauses after 15 seconds. Pauses are designed to give estimate on treatment length for each lid.



**NuSight Medical<sup>®</sup>**

*Dry eye relief...In the blink of an eye*

18029 Calle Ambiente, Suite 509  
Rancho Santa Fe, CA 92067  
[www.NuSightMedical.com](http://www.NuSightMedical.com)  
1-833-368-5437

\*This Guide is a supplement for training received in your doctor's office and is NOT all inclusive. Review the User Manual & Videos at [www.NuSightMedical.com](http://www.NuSightMedical.com)